

signia

Life sounds brilliant.

Hear the colors of sound.

Accessories for hearing instruments.



[signia.to/
accessories](https://www.signia.to/accessories)

Hearing
Systems

SIEMENS



Personalize your hearing experience.

Taking the kids to school in the morning, meeting friends during your lunch break, watching TV in the evening: Every day is full of different listening situations and conversations. Wouldn't it be great if you could stream sound right into your ears, discreetly adjust your hearing aids and easily recharge them?

Discover our convenient solutions on the following pages:

Audio streaming

easyTek/easyTek App	p. 6
Voicelink	p. 13
TV Transmitter	p. 13

Remote controls

miniPocket	p. 14
easyPocket	p. 15
touchControl App	p. 16


Charging station

eCharger	p. 18
Car adapter	p. 18

Practical accessories.

Keeping you connected and in control.

Overview

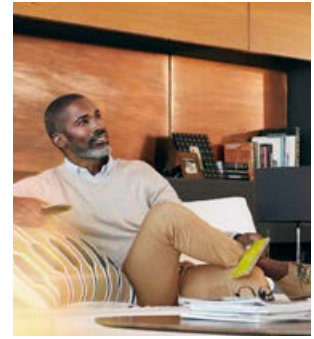

Your hearing aids help you maneuver through your day, accessories help you do it even more flexibly.

Whether you'd like to change settings remotely, stream audio directly, or take advantage of rechargeability, our broad range of stylish, practical, and easy-to-use accessories makes everyday life easier.

The right accessories can make the functions of your hearing instruments more accessible, giving you the confidence that comes with convenient control. Imagine how freeing it would be to discreetly switch programs or change volume levels by simply tapping your smartphone, to give a microphone to a speaker so that you can hear every word, loud and clear, or to simply place your hearing aids in a charger overnight, so that you're ready for each new day.

easyTek and easyTek App



touchControl App

TV Transmitter



VoiceLink

easyPocket



miniPocket



eCharger

easyTek and easyTek App.

Enhancing your listening experience, convenience and discretion.

Imagine the clarity of a phone call, TV show, or music streamed directly to your ears. easyTek™ does that - and more. Worn around the neck on top of your clothing or discreetly hidden underneath, it keeps you connected and in control by streaming audio directly into your hearing aids, or acting as a remote control. And when combined with the easyTek App™ for Android and iOS smartphones, you can also use your phone to discreetly adjust programs and volume.



easyTek and easyTek App


easyTek.

Technology that's easy to use.

Transforming wireless hearing aids into a high-quality, stereo headset is simple. The smart multi-function button lets you turn easyTek on and off, answer a phone call, change listening programs or easily switch audio sources like music players, TVs and more, with just one button.

The straightforward volume controls allow you to quickly change the volume of your hearing aids. And the easy-to-read LED lights give you important information, such as when it's time to recharge the battery.

Audio streaming.

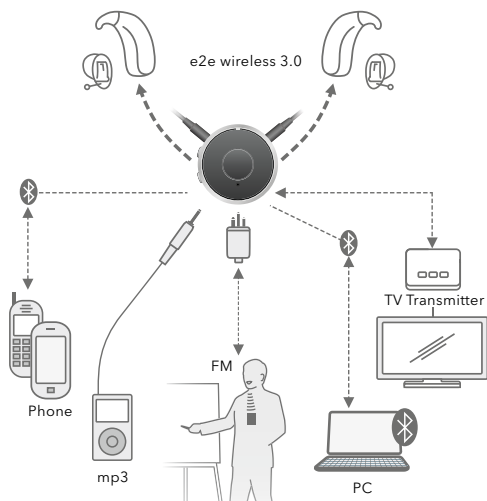

easyTek lets you connect your hearing aids to iPhones, iPads, Android devices and to any Bluetooth-enabled phones.

Audio streaming

easyTek in detail.





- 1 Multi-function button
- 2 Neckloop antenna
- 3 Microphone openings
- 4 Status information with LED indicators
- 5 Neckloop ports
- 6 Volume buttons
- 7 Port for audio devices with 3.5 mm jack
- 8 Port for FM receiver
- 9 Reset button
- 10 Micro-USB port for charging

Pairing easyTek.

As easy as 1, 2, 3.

Before you can use all advantages the easyTek has to offer, you need to pair it with your hearing instruments or other devices. Just follow the simple steps on the right.

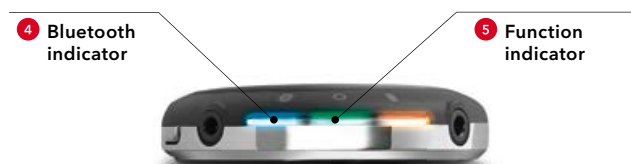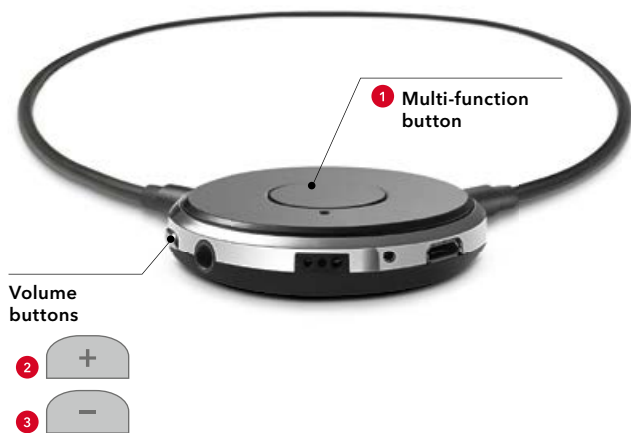

Audio streaming

Pairing easyTek with your hearing aids



Step 1:

Put the easyTek around your neck. Turn the easyTek on by pressing **1** until the green LED **5** turns on.



Step 2:

Put your hearing instruments into your ears and turn them on. The green LED **5** flashes briefly and then slowly fades. Now your hearing aids and the easyTek are paired.

Pairing easyTek with Bluetooth-enabled devices



Step 1:

Turn the easyTek on by pressing and holding **1** for several seconds. If the blue LED **4** does not automatically start flashing, press **1** and **2** until it does.



Step 2:

Turn on the Bluetooth function of the device you want to pair the easyTek with. Search for the easyTek and select it. The blue LED **4** flashes briefly and then slowly fades.

Pairing easyTek with the TV Transmitter



Step 1:

Turn the easyTek on by pressing and holding **1** for several seconds. Press **1**, **2** and **3** simultaneously until the blue LED **4** flashes.




Step 2:

Turn the TV Transmitter on within 1m of the easyTek and wait for the pairing to complete. This takes approximately 2 minutes. The blue LED **4** will stop flashing and remain on.

easyTek App.

More discretion, more convenience, more possibilities.

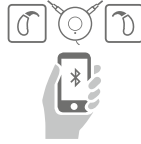

The easyTek App gives the easyTek even more functionality while adding the element of discretion.

The easyTek App lets you see the status of your hearing aids, select audio sources or make remote control adjustments. You can quickly change the volume or bass and treble while the clear icons in the program list allow you to easily choose your desired listening program. And because you can make all adjustments with your smartphone, changing your hearing aid settings simply looks like checking for a message.



Main screen

The main screen of the easyTek App allows you quick access to important adjustments.



Hearing aid functions

Thanks to the clearly arranged interface, you can easily control more precisely how you want to hear.

Extended streaming options.

For clearer hearing in noisy environments and at home.



VoiceLink

VoiceLink.

The VoiceLink™ companion microphone wirelessly connects to easyTek. When given to a speaker, it directly streams their voice to your hearing aids - which is especially helpful in difficult listening situations such as meetings, conferences or presentations.



TV Transmitter

TV Transmitter.

The TV Transmitter* connects TVs without Bluetooth functionality to the easyTek. This way, you can wirelessly stream sound directly into your hearing instruments and fully enjoy your favorite movie.

* Available as of autumn 2016.



For information on app download, see p. 17.

miniPocket.

Discreet control in the palm of your hand.



The ideal solution for anyone who wants discreet, convenient control of their hearing aids without the need for a large remote control or smartphone. Reliable and easy to use, the miniPocket™ is small enough to carry on a key ring yet powerful enough to control all essential features.



miniPocket in detail.




easyPocket.

The sleek easyPocket™ with its large, convenient buttons and an easy-to-read display makes controlling the features of your hearing instruments easier than ever before.

easyPocket

touchControl App.

Simply discreet.

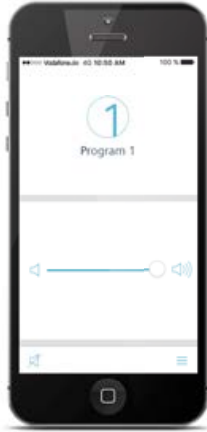


Turn your smartphone into a discreet remote control for your hearing aids: The touchControl App™ lets you enjoy a high level of handling comfort right at your fingertips.



Program and volume changes

Adapt to changing listening situations by making easy and discreet adjustments.

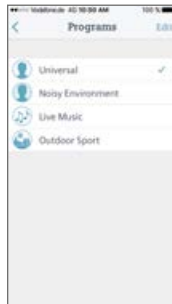

Program list

The program list uses clear icons with text to allow you to easily choose your desired program.

App download.

App-solutely easy to get!

Easy to use and easy to get: Simply download the easyTek App or the touchControl App to your Android or iOS smartphone on the Google Play Store or Apple App Store. The best part is: The download is free!

Step 1:



Make sure you have a Google Play Store account for your Android phone or an Apple App Store account for your iPhone.

Step 2:



Log into your Google Play Store or Apple App Store account with your Android or iOS smartphone. Find the easyTek App or touchControl App.

Step 3:



Follow the instructions to install the app on your smartphone.

Step 4:



Discover how easy it is to control your hearing instruments by simply trying your new app. If you should have any questions, please do not hesitate to ask your Hearing Care Professional for support.

Charging station

eCharger.

Never run out of power.



eCharger

Refresh your hearing instruments: The eCharger recharges and dehumidifies your instruments overnight, so you don't need to worry about changing batteries.

Car adapter.

Your eCharger goes mobile.



Car adapter

The optional car adapter lets you take your eCharger with you even when you're on the road so you can top up your hearing aid batteries wherever you go.



If you are not sure which accessories are compatible with your hearing instruments, your Hearing Care Professional will be glad to help.

The information in this document contains general descriptions of the technical options available, which do not always have to be present in individual cases and are subject to change without prior notice.

The Bluetooth word mark and logos are owned by the Bluetooth SIG, Inc. and any use of such marks by Signia GmbH is under license. Other trademarks and trade names are those of their respective owners.

Android and Google Play are trademarks of Google Inc.
Apple App Store is a trademark of Apple Inc.

Legal Manufacturer

Signia GmbH
Henri-Dunant-Strasse 100
91058 Erlangen
Germany

Signia GmbH is a Trademark Licensee of Siemens AG.

Order No. 02922-99C1-7600 | Printed in Germany
© 07.2016 Signia GmbH

Find us on:
signia-hearing.com

facebook.com/signiahearing

youtube.com/signiahearing